

Study Spotlight: ZEISS CT LUCIA 611P/PY – Long-term clinical outcomes



Seeing beyond

Scanning Electron Microscope Analysis (SEM) and Long-term Clinical Results with the ZEISS CT LUCIA 611P/PY

Source



Title

Long-term clinical results and scanning electron microscopic analysis of the aspheric, hydrophobic, acrylic intraocular lens CT LUCIA 611P(Y)



Authors

Dr. Andreas F. Borkenstein
Dr. Eva Maria Borkenstein



Publication


Clinical Ophthalmology Dove Press Journal: July 5, 2018

Methodology

CT LUCIA 611P/PY

- Patients over 50 years of age – mean age 75.6 ± 12.3 years
- Postoperative assessment of visual acuity, slit lamp examination with the focus on glistening and evaluation of posterior capsule opacification 1-Day, 1-Week, 4-Weeks, 3-Months and 1-Year after surgery
- SEM-analysis of optic-haptic junction and 360° square edge

Sample Size

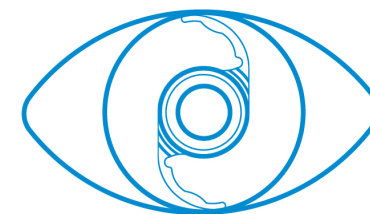

96 eyes of 54 patients

Results

The results of this long-term observational study demonstrate the safety and efficacy of the IOL ± 0.50 D for 91.7% and mean 0.02 LogMAR (range 0.14 to – 0.10)

- BCDVA increased from mean 0.48 LogMAR preoperatively to mean 0.02 LogMAR (range 0.14 to – 0.10) 1 year after surgery.
- The spherical equivalent was within ± 0.50 D for 91.7% of the subjects and the target refraction of ± 1.00 D was achieved for all study subjects. No changes in refraction were observed between 1 and 12 months.
- 90.5% (38 of 42) of the subjects, who underwent bilateral surgery with implantation of the IOL, never required glasses for distance again.
- The wider, thicker optic–haptic transition of the IOL resulted in significantly increased stiffness.
- This enabled improved centering of the IOL and enhanced rotational stability, refractive predictability and PCO prevention.
- 3 cases of PCO requiring Nd:YAG were reported. No significant PCO necessitating an intervention (laser capsulotomy) 1 year after surgery was detected in the remaining eyes.
- For the ZEISS CT LUCIA 611P/PY, no glistening was reported up to 1 year after surgery.